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## ENG – Bacterial vaginosis test

- Collect and send the sample at same day (Monday to Thursday). The sample shouldn't be taken on a friday or weekend.
  - Please keep in mind that the sample should not be collected before an extended holiday period when the mail will be slower than usual. Drop off the parcel at a post office to keep the sample well-preserved.
1. Take out the transport tube, remove the blue cap, and set the tube upright on a table. Be careful not to let the tube fall over and spill the liquid.
  2. Open the protective pouch at the shaft end of the sampling swab. Take hold of the shaft and pull the swab out of the pouch. Do not touch the absorbent sampling end of the swab with your fingers.
  3. Insert the swab about 5 cm into your vagina and rotate it so that the absorbent end rubs against the vaginal wall for approximately 30 seconds. Remove the swab from your vagina but do not put it down.
  4. Insert the absorbent end of the swab into the liquid inside the tube. Bend the shaft to break it off at the (red dotted) score line. Leave the sampling end of the swab in the tube. The shaft end can be discarded. The sampling end will stick out slightly from the tube after it is broken off. Press the swab down with the cap. Screw the cap on securely. Turn the tube upside-down to make sure it will not leak.
  5. Affix the barcode label supplied in the package lengthwise to the transport tube.
  6. Place the tube in the resealable plastic bag containing a piece of absorbent paper and seal the bag carefully. Record the date of sampling on the label affixed to the bag.
  7. Close the box carefully by inserting the wing-lets into the slots in the corners. Send the box in via regular mail. The postage is prepaid and no stamp is needed.

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### CHECKING THE ANALYSIS RESULT

Once your sample has been analysed, you'll receive a text message. You can check your results from the **testaakotona.fi** website after you have logged in with your bank ID.