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ENG – Helicobacteria test

- If you wish to confirm the results of the helicobacter treatment, please wait four weeks after the treatment before taking the sample.
- It is important to note that some medications may affect the reliability of the test. For this reason, any acid blocker treatment must be stopped two weeks before testing for helicobacter pylori and any antibiotics must be stopped four weeks before the testing.
- We recommend collecting the sample in the morning and sending it on the same day. If you collect the sample in the evening, store it in the fridge overnight.
- Deliver the package to a post office. To ensure proper storage, do not leave the box in an outside mailbox.

1. Set the disposable wrap in the packaging on the toilet as instructed in the picture. Defecate on the wrap.
2. Open the sample container. Place approximately one teaspoon of feces in the container using the spatula attached to the lid of the container. Close the cap tightly.
3. Place the sample in the ziplock bag containing a piece of absorbent paper and be sure to close the bag all the way.
4. Record the sampling date on the sticker on the bag.
5. Place the sample in the delivery box and close the box carefully by folding the small wings into the holes on the sides of the box.
6. Deliver the package to a post office as soon as possible. The postage has been paid in advance.

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CHECKING THE ANALYSIS RESULT

Once your sample has been analysed, you'll receive a text message. You can check your results from the **testaakotona.fi** website after you have logged in with your bank ID.