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ENG – Yeast infection test

- Collect and send the sample at same day (Monday to Thursday). The sample shouldn't be taken on a Friday or weekend. If you take the sample in the evening, store it in a refrigerator overnight. Please keep in mind that the sample should not be collected before an extended holiday period when the mail will be slower than usual.
 - Drop off the parcel at a post office. To keep the sample well-preserved, do not leave the parcel in a post box outdoors.
1. Remove the gel transport tube from the carton and the protective pouch. Hold the white cap and twist it so that it comes off at the dotted line. This cap can be thrown into the trash.
 2. Take hold of the blue cap attached to the sampling swab and pull the swab out of its protective pouch. Do not touch the cotton part at the sampling end of the swab with your fingers.
 3. Insert the swab about 5 cm into your vagina and rotate it so that the cotton end rubs against the vaginal wall for approximately 30 seconds. Remove the swab from your vagina but do not put it down.
 4. Push the cotton end of the swab into the transport tube gel until the blue cap is securely in place.
 5. Affix the barcode label supplied in the package lengthwise to the transport tube.
 6. Place the tube in the resealable plastic bag containing a piece of absorbent paper and seal the bag carefully. Record the date of sampling on the label affixed to the bag.
 7. Put the sampletube in the shipping box. Close the box carefully by inserting the wing-lets into the slots in the corners. Send the box in via regular mail. The postage is prepaid and no stamp is needed.

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CHECKING THE ANALYSIS RESULT

Once your sample has been analysed, you'll receive a text message. You can check your results from the **testaakotona.fi** website after you have logged in with your bank ID.