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## ENG – HPV test

- The sample collection swab is disposable and you cannot bend or remove the white part of the sampling swab before collecting the sample.
  - Mail the sample on the day following the sample collection at the latest.
  - Do not collect the sample while you are menstruating.
  - If you are using vaginal medication, contraceptives, oestrogen products or other similar vaginal products, avoid the use of these products for six hours before collecting the sample.
1. Take out the sampling swab. Hold it from the blue handle. Do not touch the white end of the sampling swab with your hands. Keep the test swab level during the sample collection.
  2. Insert the test swab as deep in the vagina as you can (approx. 10 cm) and turn the swab around for one full circle. Take the swab out (keep it level).
  3. Let the swab dry for 3–10 minutes so that the swab is against the tube and the white part is in the air.
  4. Insert the white part of the sampling swab in the tube and bend the test swab so that the white part falls in the tube. Avoid touching the white part with your fingers and hands.
  5. Close the tube's cap carefully and dispose of the blue part of the sampling swab.
  6. Set the tubes in the plastic ziplock bag in the package and close the bag tightly.
  7. Close the box carefully by inserting the wing-lets into the slots in the corners. Send the box in via regular mail. The postage is prepaid and no stamp is needed.

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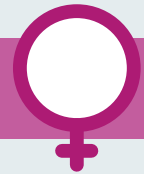
### CHECKING THE ANALYSIS RESULT

Once your sample has been analysed, you'll receive a text message. You can check your results from the **testaakotona.fi** website after you have logged in with your bank ID.

If high-risk papilloma virus is not detected in your sample, you have no signs of risk of cervical cancer.

If a high-risk papillomavirus is detected in your sample, we recommend that you seek further investigation at a gynecologist.