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## ENG – IBD test

- We recommend collecting the sample in the morning and sending it on the same day. If you collect the sample in the evening, store it in the fridge overnight.
  - To ensure proper storage, do not leave the box in an outside mailbox.
1. Set the disposable wrap in the packaging on the toilet as instructed in the picture. Defecate on the wrap.
  2. Place at least one teaspoon and at the most half of the container of feces in the container using the spatula attached to the lid of the container.
  3. Carefully close the sample container and ensure that the sample does not come out of the container. Place the sample in the ziplock bag containing a piece of absorbent paper and be sure to close the bag all the way.
  4. Record the sampling date on the sticker on the bag.
  5. Place the sample in the delivery box and close the box carefully by folding the small wings into the holes on the sides of the box.
  6. Deliver the package to a post office. The postage has been paid in advance.

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### CHECKING THE ANALYSIS RESULT

Once your sample has been analysed, you'll receive a text message. You can check your results from the **testaakotona.fi** website after you have logged in with your bank ID.

If elevated levels of calprotectin is not detected in the sample, the study does not provide any indication of inflammatory bowel disease.

If an elevated calprotectin content is detected in the sample, the result gives a reason for further investigation by physician.