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ENG – Iodide test

- Extreme fluctuations in fluid balance diminish the reliability of the result. Avoid drinking copious amounts of fluids and strong physical exertion for a period of a few hours before sample collection.
 - Mail the sample at same day as you collect it. If you take the sample in the evening, store it in a refrigerator overnight.
 - Eaten foods do not affect the result of the test, so sampling does not require special diet.
1. Wash your hands and urinary opening with water (do not use any cleansers) before sample collection.
 2. Take the single-use cup from the box and gently press it by its sides in order to straighten its bottom. Less than half a cup of urine will provide a sufficient sample amount.
 3. Place the cup on a level surface, such as the edge of a washbasin. Remove the cap of the sample tube included in the box and take out the pipette from its protective bag. Suck up urine into the pipette: first press the bulbous end, then release the pressure with the tip of the pipette inserted into the cup containing urine.
 4. Use the pipette to transfer urine into the sample tube so that the tube is filled up to the 10ml level at minimum. Pour the rest of the urine into a toilet bowl and throw the cup into a rubbish bin.
 5. Screw on the cap of the sample tube tightly and ensure that it does not leak. Put the sample tube in the small plastic bag included in the package. The plastic bag includes a piece of blotter. Close the bag tightly and place it in a packaging box.
 6. Close the box carefully by inserting the wing-lets into the slots in the corners. Send the box in via regular mail. The postage is prepaid and no stamp is needed.

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CHECKING THE ANALYSIS RESULT

Once your sample has been analysed, you'll receive a text message. You can check your results from the **testaakotona.fi** website after you have logged in with your bank ID.