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ENG – Threadworm test

- The sample is collected immediately upon waking up, strictly before defecation or washing.
 - The sampling swab is disposable and it should be placed in the tube as a whole after the sample collection. Make sure the tube is carefully closed.
 - The package includes two disposable sampling swabs, each in their own tube. Threadworms do not lay eggs every night so we recommend taking the samples on different mornings.
 - Remember to take care of good hand hygiene after collecting the sample to prevent any possible threadworm infections.
1. Packaging contains a disposable sampling stick in its own tube. If you so wish, you can dip the fluffy end of the sampling swab in tap water before collecting the sample.
 2. Rub the sampling swab quite strongly against the skin around the anus and insert the end of the swab in the anus to a depth of approximately 0.5 cm.
 3. Insert the swab back into the tube. Check that the cap is tightly closed and place the tube in the plastic ziplock bag in the package.
 4. Take another sample same way as the first one. Place the sample tube from the following morning in the same bag. Check that you have closed the plastic bag carefully.
 5. Close the box carefully by inserting the wing-lets into the slots in the corners.
 6. Send the box in via regular mail. The postage is prepaid and no stamp is needed.

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CHECKING THE ANALYSIS RESULT

Once your sample has been analysed, you'll receive a text message. You can check your results from the **testaakotona.fi** website after you have logged in with your bank ID.