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ENG – Lactose intolerance test

- Do not drink coffee for an hour before collecting the sample.
 - Rinse your mouth with water before collecting the sample.
1. The package includes a disposable sampling swab and a tube. Take out the swab without touching the sample collection end.
 2. Rub the sampling end of the swab against the inside of your cheek on the both sides, turning the swab around strongly around 20 times.
 3. Let the swab dry with the sampling end upwards for 15-30 minutes, for instance against the tube.
 4. Insert the dried sampling swab to the tube. Do not touch the sampling end of the swab with your hands.
 5. Break the sampling swab at the thinner breaking point. Close the cap.
 6. Place the sample casing in the plastic ziplock bag in the packaging and close the bag carefully.
 7. Place the samples in the delivery box and close the box carefully by folding the small wings into the holes in the sides of the box. Mail the box either. Postage has been paid in advance.

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CHECKING THE ANALYSIS RESULT

Once your sample has been analysed, you'll receive a text message. You can check your results from the testaakotona.fi website after you have logged in with your bank ID.