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ENG – Salmonella test

- Salmonella test is delicate to changes in temperature and long storage times. The salmonella sample is collected to the bacteria delivery tube that contains a liquid preservative. It is important that the faecal sample is completely in the liquid preservative and that the sample is mailed to the laboratory as soon as possible after the collection.
 - We recommend collecting the sample in the morning and sending it on the same day. If you collect the sample in the evening, store it in the fridge overnight.
1. Check that the packaging and the included separate barcode have the same code.
 2. Set the disposable wrap in the packaging on the toilet as instructed in the picture. Defecate on the wrap.
 3. Open the protective packaging of the transport tube. Pick up the sampling swab from the protective packaging and insert the swab's end in the faeces so that faeces will be collected on the swab as instructed in the picture.
 4. Open the cap of the transport tube, set the swab on the bottom of the tube and break the swab against the mouth of the tube. Close the cap tightly by twisting it. The broken end of the sampling swab will be adhered to the cap.
 5. Attach the barcode sticker included in the package to the transport tube vertically. Record the sampling date on the sticker on the bag.
 6. Insert the tube in a small plastic bag included in the package. The plastic bag includes a piece of blotter. Close the bag tightly. Set the bag in the transport box so that the tube is in the gel bag's groove. Place the yellow thermal insulation lid on the box.
 7. Close the cardboard box carefully by folding the wings inside. Mail the sample as soon as possible by bringing it to the post office. Sample should not be left in the mailbox. Postage has been paid in advance.

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CHECKING THE ANALYSIS RESULT

Once your sample has been analysed, you'll receive a text message. You can check your results from the **testaakotona.fi** website after you have logged in with your bank ID.