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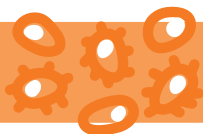
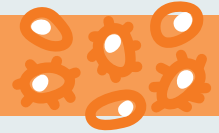
ENG – Bowel cancer test


- The sample collection package includes three test tubes where samples will be collected at separate defecation times. Bleeding in the intestines may be periodic, which means collecting the sample from three different defecation times increases the reliability of the result.
 - The test tubes in the package include a dilution liquid, do not discard it.
 - Store the sample in a refrigerator until you have collected all three samples. The sample will preserve in the dilution liquid tube for 7 days.
 - Mail the sample at the day you hve collected the last sample. Deliver the package to a post office. To ensure proper storage, do not leave the box in an outside mailbox. Please note that you should not collect the samples right before long holidays when the post runs slower than normally.
 - The sample cannot be collected when you are menstruating.
 - Eaten foods do not affect the result of the test, so sampling does not require special diet.
1. Set the disposable wrap in the packaging on the toilet as instructed in the picture and defecate on the wrap.

sample is mixed with the liquid. Once you have inserted the sample in the tube, you can dispose of the wrap by flushing it down the toilet.
 2. Open the test tube. Collect the sample with the swab attached to the cap by turning the swab in different parts of the faecal sample so that faeces will remain in the holes on both sides of the swab. Wipe excess faeces off with toilet paper. The sample is sufficient once both holes have faeces as shown in the image below. If the swab has too much faeces, it will hinder the test process.
 3. Place the swab in the test tube and close the cap. Shake the tube strongly for at least 10 seconds so that the
 4. Insert the test tube in a ziplock bag and close the bag and place it in a packaging box.
 5. Record the sampling date on the sticker on the bag.
 6. Close the box carefully by inserting the wing-lets into the slots in the corners. Send the box in via regular mail. The postage is prepaid and no stamp is needed.

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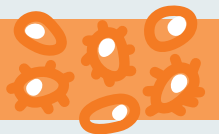CHECKING THE ANALYSIS RESULT

Once your sample has been analysed, you'll receive a text message. You can check your results from the **testaakotona.fi** website after you have logged in with your bank ID.